



Independence enabling
Sea Access Device
for People with Disabilities
and Mobility Issues





- ✓ Offers people with disabilities and mobility issues **unassisted sea access**.
- ✓ Encourages **integration and inclusion** of all members of society.
- ✓ A technological solution that promotes **quality and wellness** in everyday life.
- ✓ Assists **escorts / caretakers / families** of people with disabilities.

Parts:

- Tower
- Modular tracks
- Handrail
- Moving carriage with custom seat
- Solar panel
- Remote Control
- Appropriate labeling

Extras

- Shower
- Light alarm system
- Telemetry package

Main Features:



Autonomous sea access

Self-diagnosis system for Long distance troubleshooting



Environmentally friendly

- Use of renewable energy sources
- Does not permanently alter the beach



INNOVATION

Artificial Intelligence to protect privacy

How it operates:

- 1 Position the wheelchair next to the SEATRAC seat.
- 2 Move yourself on the seat and position your body in the center with your back resting on the back of the seat.
- 3 Place your feet on the footrest of the seat.
- 4 After checking the track is free, use the remote control (left button) to reach the sea.
- 5 The seat stops automatically and the ending point of the tracks (handrail) so that you can get off the seat and start swimming.
- 6 Use the handrail and the buoyant force of the water to help you get in and out of the sea.
- 7 After swimming, place yourself back on the seat, in the same position as when you entered the sea.
- 8 Use the remote control to return to the beach (right button) after checking the track is free.
- 9 The seat stops automatically at the starting point, if there is no other command.

Remote Control

LEFT button

to move towards the **SEA** (ending point)

RIGHT button

to move towards the **BEACH** (starting point)

To **STOP** at any time, press **EITHER BUTTON** continuously for **2 seconds**. All remote controls work with all SEATRAC devices.



Patents

Greece (Patent No.1008398)
European Union (Patent No. EP 2539509)
USA (Patent No. US 9,127,411 B2).

Conforms to all applicable European Union directives and standards and has received a CE conformity marking.

Conforms to EU directives 2006/42/EC, 2004/108/EC, 1999/5/EC, Council Recommendation 1999/519/EC, according to the relevant European harmonized standards.

Awards



Services

Our website will provide updated listings of where to find seatrac locations.

What does SEATRAC do?

- ✓ Offers people with disabilities and mobility issues unassisted sea access.
- ✓ A technological solution that promotes equality and wellness in everyday life.
- ✓ Encourages integration and inclusion of all members of society.
- ✓ Assists escorts / caretakers / families of people with disabilities.

Learn more about seatrac™

- Visit www.seatracusa.com
- Learn about seatrac™ models
- Locate seatrac™ beaches
- Find out about seatrac™ accessories
- See how seatrac™ works

Helpdesk

Free service for all end users and clients

Support Services
support@seatracusa.com
09:00-05:00, EST Monday-Friday
tel. 1.201-790-6924

Maintenance and reinstalling package

After the first year of use.



Find us at:
www.seatracusa.com



North America, Central America, South America, The Caribbean, Israel



1.201.790.6924



info@seatracusa.com