
www.seatracusa.com





Independent sea access for people with mobility issues

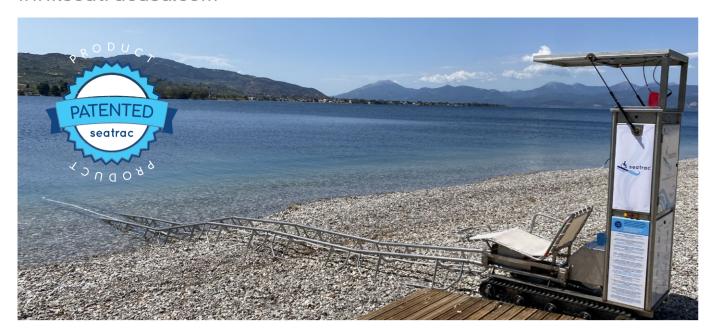



Brief Description

Offers people with disabilities and mobility issues **independence and unassisted sea access.**

Easy to install.

No more than 15 minutes for someone to place it on the beach or dismantle it and store it in the changing/storage room.


It can be removed from the beach without dismantling it and no one needs to get wet.

Ideal for tidal changes greater than 18"/.5M & sudden changes in weather conditions.



Main Features



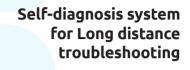





Environmentally friendly

Use of renewable energy sources

Does not permanently alter the beach



Remote control



Safety strobe alert light



Ergonomic seat



Easy to install



Mobile



seatrac mover specifications

Storage mode
Operational mode
Track section length
Track weight
Voltage
Batteries capacity
Charging options

Load capacity Embedded chair


Main features/operations

80"x 31.49"x 74.08"/ 2.05m x 0.8m x 1.9m (LxWxH) 773.62"x 31.49"x 83.37"/19.65m x 0.8m x 2.27m (LxWxH) 59"/1.5m


22LB/10kg 24VDC

- 24V-100Ah
 Embedded solar panel, or
- Plug in to the grid 220VAC/110VAC
- Travel speed 5.9"/.15m second
- Chair height at end of track in water 30" / .75m
- The device with its tracks has the ability to move itself along various ground types
- It operates via remote control by the user or the operator

seatrac mover parts

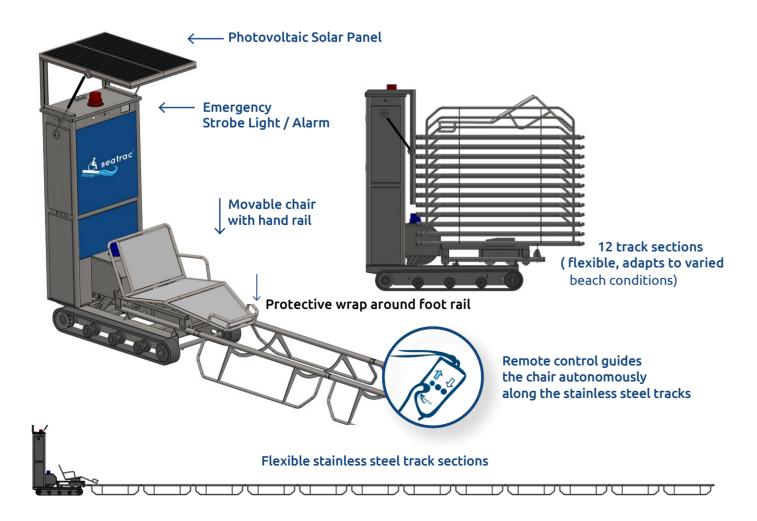

Optional: Storage / Changing room

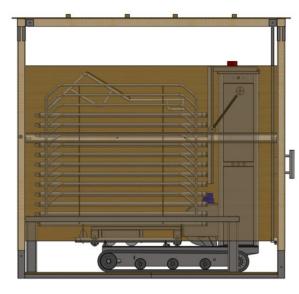



seatrac™ mover fits completely into storage room with lockable door to protect against vandalism

A day with **seatrac** mover

1 Starting the day

- 1. Access the device from storage
- 2. Check if it is charged
- 3. Move it to the beach
- 4. Untie the track sections
- 5. Assemble the track sections
- 6. Unfold the solar panel
- Drive it to the appropriate depth of about 30" / .75m with chair position at end of track with color coded handrail in water



During the day

- a. Check tide levels several times during the day
- Adjust seatrac mover to right depth

3 At the end of the day

- a. Disassemble the device
- **b.** Move it it to storage room
- c. (Optional) Plug in charger

Services

TOBEA, with end user information and feedback about satisfaction, created services that improve people's visit to the beach.

seatrac Preview

Locate your next **seatrac** experience

- Receive information on the state of all installed SEATRAC devices via email.
- You can register here www.seatracusa.com

Helpdesk

Free service for all end users and clients

Accessible Beaches Network

- Find the map in www.seatracusa.com
- Waves, crowds, unclear water, wind... depending on your preferences, you can visit the beach that suits you!
- Discover the beach that suits your needs depending on the available amenities.

Free support service support@seatracusa.com

09.00-5.00, EST Monday - Friday tel.: 1.201.790.6924



Accessibility Chain







Beach Track Re



Restroom



Changing room (collapsible, non-permanent structure)



Shaded Area



More information in www.seatracusa.com



Founded in 2012

The vision of the TOBEA team is to improve the quality of life of people with disabilities though innovative ideas and products. The aim of the company is to design, build and promote engineering applications that help society.

The company complies with the following quality management systems: ISO 9001:2015 - ISO 14001:2015 - ISO 45001:2018

Patents

(Hellenic Federation

of Enterprises)

- ✓ Greece (Patent No.1008398)
- ✓ European Union (Patent No. EP 2539509)
- USA (Patent No. US 9,127,411 B2).
- ✓ Community Trade Mark identification no. 012587887 registered 25/11/2014
- Industrial design no. 003851039-0001/003851039-0002 and 001463871-0001.

Awards






07







North America, Central America, South America, Caribbean, Israel

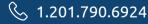

☑ info@seatracusa.com

www.seatracusa.com

TOBEA WEST, INC.